SAFE-ST IMPRESSION DISINFECTION LIQUID

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 **DEHP**™ **DE Healthcare Products**
DEDICATED TO EXCELLENCE

0°C
32°F

40°C
104°F

 **CB Healthcare Consulting GmbH**
(manufacturer)
Am Neumarkt 34
22041 Hamburg, Germany
Tel: +49 40 656680
Email: info@cbhealthcare-intl.com

CE 0482

Rev 2019/12

EN: Area of application: Disinfection of all dental impression materials. **Instructions for use:** Immerse the respective impression into the ready-for-use disinfectant solution, and remove again after the recommended application time. The impression must be fully covered by the solution. Only immerse alginate and hydrocolloids for a short time and let them dry. Then rinse the impression with water. Do not mix with cleaning agents or other disinfectants. Depending on the degree of contamination, the working solution is to be replaced (FKI recommendation: at least once per working day). The working solution can be disposed of as sewage water. **Application times:** Bacteria [incl. MRSA] as per VAH, EN 13624 and EN 14562 [dirty conditions]. Conc. 1 min.; Limited virucidal [Enveloped viruses] as per VAH, EN 13727 and EN 14561 [dirty conditions]. Conc. 1 min.; Yeast/cidal [Candida albicans] as per VAH, EN 13624 and EN 14562 [dirty conditions]. Conc. 1 min.; Limited virucidal [Enveloped viruses] as per VAH, EN 13727 and EN 14561 [dirty conditions]. Conc. 30 min.; Rotaviruses as per BGA/ DWV [clean conditions]. Conc. 1 min. **Safety guidelines:** Harmful to aquatic life with long lasting effects. Avoid release to the environment. Dispose of contents / container in accordance with all local, regional, national and international regulations. **Directive 648/ 2004/ EC:** Contains < 5 % cationic surfactants, fragrance. **100 g of solution contain:** 0,24 g of benzyl C-12-16 alkyl dimethylammonium chloride, 0,24 g of N, N-didecyl-N, N-dimethylammonium chloride, 0,24 g of C12-14-alkyl [ethylenyl] methyl dimethylammonium chloride, 0,35 g chlorhexidine gluconate.

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