

988 4681

# CYBERTECH

**ULTRACLEAN LEMON**
**SURFACE  
DISINFECTION  
WIPES**
**REFILL**
**Lingettes désinfectantes à action rapide**
**Schnelldesinfektionstücher**
**Desinfectiedoekjes**
**Utěrky pro rychlou dezinfekci**
**Inawilżanedo szybkiej dezynfekcji**

**EN:** Type of application: Ready-to-use alcoholic wet wipes for the prophylactic surface disinfection of alcohol-resistant noninvasive medical devices. Notes on application: Open top end of the bag with the impregnated wipes and put it in the intended disposal tub. Pull the wipe in the middle of the roll and thread it through the star in the lid from below. Push the lid onto the tub and close it tightly. Carefully wipe the area to be disinfected and let the disinfectant take effect. Use only on alcohol-tolerant materials (e.g. not suitable for acrylic glass). Once the bag has been opened, the wipes should be used within 28 days in suitable containers. The opening date, best before date, as well as the legally-relevant notes regarding the impregnating fluid are to be observed and transferred to a label for the external identification of the container. After removing a wipe, close the tub carefully and tightly. The disposal tub is to be prepared in accordance with the adjacent preparation guidelines, using a suitable disinfection product. Application times (of the impregnating fluid): Bacteria according to EN 13727, EN 1276, EN 13697 (dirty conditions): Conc. 1 min. *Candida albicans* according to EN 13624, EN 1650, EN 13697, EN 16615 (dirty conditions): Conc. 1 min. TB pathogen (*M. terrae*) and mycobacteria (*M. avium*) according to EN 14348 (dirty conditions): Conc. 1 min. "Limited virucidal PLUS" (enveloped viruses, e.g. HBV, H1N1, HCV) as well as noro-, adeno- and rotaviruses according to EN 14476 (clean conditions): Conc. 1 min. Poliovirus (SV40) (according to RKI/DW with pollution) Conc. 1 min. Application times for impregnated wipes: Bacteria and *Candida albicans* according to EN 1661 and IAH, dirty conditions: Conc. 5 min. Contents: 120 cloths made of PP/MS blended fabric, 12 cm x 20 cm, 100 g of fluid contains: 20 g 1-propanol, 5 g 2-propanol. Safety guidelines: H229 Flammable solid, H319 Causes serious eye irritation, H336 May cause drowsiness or dizziness. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Wear eye protection / face protection. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists: Get medical advice/attention. Dispose of contents/ container in accordance with local/regional/national/international regulations. Keep out of reach of children. Read identification label carefully before use. Flash point as per DIN 51755: approx. 29°C. Application max. 50ml per m<sup>2</sup>. The total applied amount per room must not exceed more than 100ml per m<sup>2</sup> of room area. Keep the containers tightly sealed. Only put the packaging into the recycling system if it is empty of residue. Dispose of residual product in accordance with EAK 07 06 99. Directive 648/2004/EC. Contains: Fragrances



Danger  
Danger  
Gefahr  
Gevaar  
Nebezpečí  
Niebezpieczeństwo



(01)04062183028192



LOT



UFI: WW0U-0X2W-W109-C3GY



Medical Device /Dispositif medical  
Medizinprodukt/Medisch hulpmiddel  
Zdravotnický prostředek/ Wyrób medyczny



(manufacturer)

**CB Healthcare Consulting GmbH**
**Am Neumarkt 34**
**22041 Hamburg, Germany**
**Tel: +49 40 656680**
**Email: info@cbhealthcare-intl.com**

Rev.2021/02

**120**


/00/20

