## RAPID SURFACE DISINFECTION, LEMON

**Ready-to-use, aldehyde-free rapid disinfectant for a rapid and effective disinfection**

- **Alcohol based**
- **Residue-free**
- **Fast acting**
- **Avoids any discolorations of surfaces**

### PRODUCT DESCRIPTION:

Ready-to-use, aldehyde-free rapid disinfectant based on ethanol for an effective disinfection of medical inventory, medical devices as well as other kinds of surfaces. Especially suitable for difficult to reach surfaces. The aldehyde-free formulation avoids any discolorations of surfaces after the use of CyberTech Rapid Surface Disinfection, Lemon.

Do not use on alcohol-sensitive surfaces (e.g. acrylic glass).

### AREAS OF APPLICATION:

Acc. to Medical Device Directive (MDD): quick disinfection and cleaning of alcohol-resistant, non-invasive medical devices. Acc. to Biocidal Products Regulation (BPR): quick disinfection and cleaning of alcohol-resistant surfaces of all kinds.

### SPECTRUM OF APPLICATION AND EXPOSURE TIME:

Surface disinfection (wipe disinfection), bactericide, yeasticidal as per VAH: 1 min.; tuberculocide (M. terrae): 1 min; limited virucide (including HBV, HIV, HCV), rotaviruses: 30 sec.; noroviruses (MNV): 1 min.

### APPLICATION:

Spray surfaces and objects with CyberTech Rapid Surface Disinfection, Lemon from a distance of at least 30 cm until entirely moistened. After exposure time, wipe off surfaces with a clean single-use cloth if required. To avoid the formation of alcoholic fumes, CyberTech Rapid Surface Disinfection, Lemon can be sprayed on a disposable tissue and used for wiping of surfaces by respective wetting. CyberTech Rapid Surface Disinfection, Lemon does not leave any residues on surfaces. Do not use on alcohol-sensitive surfaces (e.g. acrylic glass).

### COMPOSITION:

100 g ready to use solution contains: 45 g ethanol (CAS: 64-17-5).

### SPECIAL INFORMATION:

Spray surfaces and objects with CyberTech Rapid Surface Disinfection, Lemon from a distance of at least 30 cm until entirely moistened. After exposure time, wipe off surfaces with a clean single-use cloth if required. To avoid the formation of alcoholic fumes,

CyberTech Rapid Surface Disinfection, Lemon can be sprayed on a disposable tissue and used for wiping of surfaces by respective wetting. CyberTech Rapid Surface Disinfection, Lemon does not leave any residues on surfaces.

### PRODUCT STATUS/LISTINGS:

Conform to the EU Biocidal Product Regulation (BPR, 528/2012). Conform to the EU guideline 93/42/EC for medical devices.

### AVAILABLE DELIVERY FORMS

Itemnumber	Packaging
9882787	1L Spray bottle
9882788	5L Canister

### AVAILABLE ACCESSORIES:

Itemnumber	Packaging
9796277	Sprayhead for 1 Liter Bottle
9796276	Tab/dosing device for 5L canister
9796278	Canister-key
9796272	Dosing pump for 5L Canister

Use disinfectants safely. Always read the label and product information before use.

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