# System Dispenser Wipes

- A - Dispenser bucket
- B - System Wet Wipes
- C - System Wet Wipes non-alcoholic

## ■ A - Profection Dispenser bucket

### PRODUCT DESCRIPTION:

For filling with non-woven wipes and disinfectants/cleaner.

### IMPORTANT:

Before refilling the dispenser with lid must be cleaned and has to be disinfected with a suitable disinfectant. During refilling procedure it is recommended to document the kind of solution, its concentration rate as well as the preparation date on the attached sticker.

### SHELF LIFE:

28 days

### SMALL:

Itemnumber	Packaging
9794941	Profection System Dispenser small for size 25x25cm

**Application:** Insert the wipes and soak with 1.5 litres of ready-to-use disinfectant solution. After 30 minutes of soaking, the wipes are ready to use. Close the dispenser after use.

### LARGE:

Itemnumber	Packaging
9794933	Profection System Dispenser large for size 32x30cm

**Application:** Insert the wipes and soak with 2.5 litres of ready-to-use disinfectant solution. After 30 minutes of soaking, the wipes are ready to use. Close the dispenser after use.

# System Dispenser Wipes

- A - Dispenser bucket
- B - System Wet Wipes
- C - System Wet Wipes non-alcoholic

- B - Profection System Wet Wipes
- Quick and easy application
- Suitable for hygienically critical and near-patient areas
- Active against all enveloped viruses incl. rota- and noroviruses

#### PRODUCT DESCRIPTION:

Profection System Wet Wipes are ready to use alcoholic wipes for disinfecting near patient and hygienically critical areas. It is not necessary to wipe the area again after using Profection System Wet Wipes as it does not leave any residue on surfaces.

#### AREAS OF APPLICATION:

Acc. to Medical Device Directive (MDD): quick disinfection and cleaning of alcoholresistant, non-invasive medical devices. Acc. to Biocidal Products Regulation (BPR): quick disinfection and cleaning of alcohol-resistant surfaces of all kinds.

#### APPLICATION:

Wipe the alcohol-resistant surfaces with Profection System Wet Wipes until they are completely wet and adhere to the contact time. Profection System Wet Wipes are designated for the use with refillable wipe dispenser systems. Details for filling and processing of the wipe dispenser systems are to be regarded. Not suited for acrylic glass (PMMA).

#### COMPOSITION:

100 g ready to use solution contains: 45 g ethanol (CAS: 64-17-5).

#### SPECTRUM OF APPLICATION AND EXPOSURE TIME:

Surface disinfection (wipe disinfection), bactericide, yeasticidal as per VAH: 1 min.; tuberculocide (M. terrae): 1 min; limited virucide (including HBV, HIV, HCV), rotaviruses: 30 sec.; noroviruses (MNV): 1 min

#### SPECIAL INFORMATION:

Open the refill package. Place the contents, including bag, into the dispenser. Pull the first cloth from the middle of the roll into the lid of the dispenser and replace the lid on the container. Please refill the container only with the refill package of Disinfection Wipes. Warning: Flammable solid. If medical advice is needed, have product container or label at hand. Keep out of reach of children. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Keep container tightly closed. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Dispose of contents/ container to approved disposal company or local collection. For Professional Use Only.

#### LISTUNGEN / PRODUKTSTATUS:

Conform to the EU Biocidal Product Regulation (BPR, 528/2012).

Conform to the EU guideline 93/42/EC for medical devices.

#### AVAILABLE DELIVERY FORMS:

Itemnumber	Packaging
9795159	Profection System Wet Wipes 25x25cm
9794934	Profection System Wet Wipes 32x30cm

# System Dispenser Wipes

- A - Dispenser bucket
- B - System Wet Wipes
- C - System Wet Wipes non-alcoholic

## ■ C - Profection System Wet Wipes non-alcoholic

- Alcoholfree
- Wipes for rapid disinfection of large surfaces
- Fast acting
- High degree of material compatibility
- Easy and practical to apply

### PRODUCT DESCRIPTION:

Alcohol-free, disinfection wipes for rapid disinfection and cleaning of all kinds of surfaces in medical areas as well as in all areas are based on quaternary ammonium compounds. Profection System Wet Wipes non-alcoholic do not contain aldehydes or phenols. The high-quality wipes guarantee maximum release of active ingredients and cleaning power.

### AREAS OF APPLICATION:

Acc. to Medical Device Directive (MDD): Disinfection and cleaning of alcohol-sensitive, non-invasive medical devices. Acc. to Biocidal Products Regulation (BPR): For disinfecting and cleaning all types of alcohol-sensitive surfaces

### APPLICATION:

Thoroughly dampen surfaces and objects. After the application time has elapsed, wipe down with a disposable wipe, where necessary.

### COMPOSITION:

100 g ready to use solution contains: 0.5 g Alkyl (C12-16) dimethylbenzyl ammonium chloride; 0.25 g Didecyldimethylammoniumchloride.

### SPECTRUM OF APPLICATION AND EXPOSURE TIME:

Surface disinfection (wipe disinfection) bactericidal, yeasticidal per VAH [German Association for Applied Hygiene]: 1 min., effective against all enveloped viruses per the Robert Koch Institut recommendation incl. HBV/HIV/HCV/Vaccinia/BVDV/Influenza/flu viruses: 30 secs.

### SPECIAL INFORMATION:

Open the refill package. Shape the open package into a roll. Place the contents, including bag, into the dispenser. Pull the first cloth from the middle of the roll into the lid of the dispenser and replace the lid on the container. Please refill the container only with the refill package of Rapid Disinfection Wipes AF. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Dispose of contents/container to approved disposal company or local collection.

### LISTUNGEN / PRODUKTSTATUS:

Listed in the current disinfectant list of the VAH.

Conform to the EU Biocidal Product Regulation (BPR, 528/2012).

Conform to the EU guideline 93/42/EC for medical devices.

### AVAILABLE DELIVERY FORMS:

Itemnumber	Packaging
9794935	Profection System Wet Wipes Surface non-alcoholic 32x30cm

Use disinfectants safely. Always read the label and product information before use.

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