# SAFE-ST INSTRUMENT DISINFECTION CONCENTRATE

- **Virucidal instrument disinfection**
- **Fast acting, virucidal, liquid concentrate for disinfecting endoscopes and surgical instruments of all kinds**
- *Virucidal according to Robert Koch Institut*
- *Recommended glutaraldehyde basis concentrate*
- *Formaldehyde-free*

## PRODUCT DESCRIPTION:

Safe-ST Instrument disinfection concentrate is a fast acting, virucidal, liquid concentrate for disinfecting endoscopes and medical instruments of all kinds. The formaldehyde-free product formulation is characterised by economical working concentrations, broad spectrum effectiveness and good material compatibility.

## AREAS OF APPLICATION:

Disinfection of medical instruments and endoscopes.

## EFFECTIVE SPECTRUM AND DOSAGE:

Bactericide (including MRSA, yeastocidal: 2% - 15 min.; 1.5% - 30 min.; 1% - 60 min.; tuberculocide (M. ter-  
rae): 2% - 15 min.; 1.5% - 30 min.; 1% - 60 min. virucide including HBV/HIV, polio virus, adenovirus, papovavirus, vaccinia virus: 2% - 30 min.

## APPLICATION:

### Disinfection using the immersion bath procedure:

Clean and rinse instruments thoroughly and place them (open) into the required working solution. All surfaces and cavities requiring disinfection must be completely wetted. Rinse thoroughly (preferably with demineralised water) after the end of the contact time and dry. Should any soiling become visible in the working solution, change it immediately; otherwise, change it at least every working day. Then sterilise the instruments using a validated procedure.

**Endoscope processing:** Comply with all requirements for personal protection and the manufacturer's processing recommendations. Remove any heavy soiling from the endoscope with a lint-free cloth and immerse com-

pletely in a cleaning solution. After cleaning, rinse endoscopes thoroughly with water and immerse completely in the Safe-ST Instrument disinfection concentrate working solution. Ensure that all channels are filled and that the solution flows through them. After the contact time has elapsed, rinse the instruments thoroughly with sterile water and dry them. For further instructions, see appendix 8 of the RKI KRINKO recommendation. Requirements for hygiene during preparation of medical devices. Recommendation from the Commission for Hospital Hygiene and Infection Prevention (KRINKO) at the Robert Koch Institute (RKI) and the Federal Institute for Drugs and Medical Devices (BfArM). Bundesgesundheitsblatt 2012 · 55:1244–1310.

## APPLICATION INFORMATION:

Appropriate personal protective equipment (PPE) must be worn when handling Safe-ST Instrument disinfection concentrate. For further information about occupational health and safety, see [www.bgw-online.de](http://www.bgw-online.de). Before switching to Safe-ST Instrument disinfection concentrate, clean all instruments and baths thoroughly to remove chemical residues (e.g. aldehydes). Instrument baths must be cleaned and disinfected at least once every working day, preferably with a disposable wipe soaked in alcohol.

# SAFE-ST INSTRUMENT DISINFECTION CONCENTRATE

- **Virucidal instrument disinfection**
- **Fast acting, virucidal, liquid concentrate for disinfecting endoscopes and surgical instruments of all kinds**
- *Virucidal according to RKI*
- *Recommendation formaldehyde-free concentrate*
- *Suitable for final disinfection*

## COMPOSITION:

100g contains: 12 g glutaraldehyde.

## SPECIAL INFORMATION:

Glutaraldehyde, alkylether carboxylic acid, sulfonic acid, C14-17-sec-alkan-, sodium salt. Harmful if swallowed or if inhaled. May cause allergy or asthma symptoms or breathing difficulties if inhaled. May cause respiratory irritation. Causes severe skin burns and eye damage. May cause an allergic skin reaction. Toxic to aquatic life with long lasting effects. Wear protective gloves/protective clothing/eye protection/face protection. IF SWALLOWED: Call a POISON CENTER/doctor if you feel unwell. IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water/shower. IF INHALED: Remove person to fresh air and keep comfortable for breathing. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Immediately call a POISON CENTER/doctor. Avoid release to the environment. Dispose of contents/container to in accordance with local and national regulations.

In case of visible soiling the Safe-ST Instrument disinfection concentrate working solution must be changed immediately; otherwise, change it at least every working day.

## PRODUCT STATUS/LISTINGS:

Conform to the EU guideline 93/42/EC for medical devices.

## AVAILABLE DELIVERY FORMS:

Itemnumber	Packaging
9797758	2L bottle
9797759	5L canister

## AVAILABLE ACCESSORIES:

Itemnumber	Packaging
9796273	Dosing pump for 2L bottle
9796276	Tab/dosing device for 5L canister
9796278	Canister-key
9796272	Dosing pump for 5L Canister

Use disinfectants safely. Always read the label and product information before use.

# SAFE-ST INSTRUMENT DISINFECTION CONCENTRATE

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