SAFE-ST SURFACE DISINFECTION NON-ALCOHOLIC CONCENTRATE

- Surface disinfection concentrate non-alcoholic
- Concentrate for disinfection and cleaning of medical inventory, medical devices and other kinds of surfaces
- Aldehyde-free preparation with good cleaning properties
- Perfume-free

PRODUCT DESCRIPTION:

Safe-ST Surface Disinfection non-alcoholic concentrate is an aldehyde-free concentrate for disinfection and cleaning of medical inventory, medical devices and other kinds of surfaces. The alcohol-free formulation is based on modern quaternary ammonium compounds and contains neither aldehydes nor phenols. Safe-ST Surface Disinfection non-alcoholic concentrate is very userfriendly due to its good cleaning properties and perfume-free formulation. Safe-ST Surface Disinfection non-alcoholic concentrate can be used as an alcohol-free rapid disinfectant and is suitable for the disinfection and cleaning of acrylic glass.

AREAS OF APPLICATION:

According to Medical Device Directive (MDD): For cleaning and disinfecting non-invasive medical devices. According to Biocidal Products Regulation (BPR): For cleaning and disinfecting all types of surfaces.

SPECTRUM OF ACTION:

Bactericidal (incl. MRSA), yeasticidal, virus-inactivating (HBV/HIV/HCV/BVDV/Vaccinia/Papova viruses): 2,5 % (25g/l) – 15 min.; 2 % (20g/l) – 30 min.; 1 % (10g/l) – 60 min.

CONCENTRATION:

Surface disinfection (wipe disinfection), bactericidal, yeasticidal per VAH [German Association for Applied

Hygiene] recommendations: 2.5% (25g/l) – 15 mins.; 2% (20g/l) – 30 mins.; 1% (10g/l) – 60 mins.

Please dilute the concentration to the required solution by adding water.

	1,0 %	2,0 %	2,5 %
1 L*	10 ml	20 ml	25 ml
2 L*	20 ml	40 ml	50 ml
4 L*	40 ml	80 ml	100 ml
8 L*	80 ml	160 ml	200 ml
10 L*	100 ml	200 ml	250 ml

APPLICATION:

Surface disinfection by wiping (recommended procedure): Apply a sufficient quantity of the prepared Safe-ST Surface Disinfection non-alcoholic solution to the surfaces to be disinfected. Use mechanical action to completely wet all surfaces (scrub wipe disinfection).

COMPOSITION:

100 g contains: 6.66 g alkyl (C12-16) dimethylbenzyl ammonium chloride; 3.33 g didecyldimethyl ammonium chloride; < 5 % non-ionic surfactants, perfumes.

SAFE-ST SURFACE DISINFECTION NON-ALCOHOLIC CONCENTRATE

- Surface disinfection concentrate non-alcoholic
- Concentrate for disinfection and cleaning of medical inventory, medical devices and other kinds of surfaces
- Aldehyde-free preparation with good cleaning properties
- Perfume-free

SPECIAL INFORMATION:

Can cause severe skin burns and eye damage. Very toxic to aquatic life. Wear protective gloves/protective clothing/eye protection/face protection. IF SWALLOWED: rinse mouth. Do NOT induce vomiting. IF ON SKIN (or hair): Remove/Take off immediately all contaminated clothing. Rinse skin with water/ shower. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. IF exposed or concerned: Call a POISON CENTER or doctor/physician. Avoid release to the environment. Dispose of contents/ container to approved disposal company or local collection.

PRODUCT STATUS/LISTINGS:

Conform to the EU Biocidal Product Regulation (BPR, 528/2012).

Conform to the EU guideline 93/42/EC for medical devices.

AVAILABLE DELIVERY FORMS:

Itemnumber	Packaging
9797733	5L canister

AVAILABLE ACCESSORIES:

Itemnumber	Packaging
9796276	Tab/dosing device for 5L canister
9796278	9796278 Canister-key
9796279	Dosing pump for 5L Canister

Use disinfectants safely. Always read the label and product information before use.

SAFE-ST SURFACE DISINFECTION NON-ALCOHOLIC CONCENTRATE

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