
OCCLU® SPRAY PLUS

Okklusionsspray, grün
Occlusion spray, green
Spray d'occlusion, vert

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GB Occlu Spray Plus marks the occlusal surface with a superfine layer of micro powder. Shake and spray from a distance of 2.5 – 3 cm (1 inch). Composite, acrylic or porcelain: spray onto the antagonist; cast metal, gold or amalgam: spray directly onto the surface. In order to visualize contact points, spray directly into crowns and bridges. Can be easily removed with water. Must only be used in dental surgeries and dental laboratories.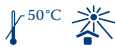

75 ml e

110 3

Made in Germany

Pressurized container. protect from sunlight and do not expose to temperatures exceeding 50 °C. Do not pierce or burn, even after use. Do not spray on a naked flame or any incandescent material. Buildup of explosive mixtures possible without sufficient ventilation. Keep away from sources of ignition – No smoking. Keep in a cool place. Keep out of the reach of children. Use only in well-ventilated areas. In case of fire, use sand, carbon dioxide or powdered extinguishing agent. Never use water. Harmful to aquatic organisms, may cause long-term adverse effects in the aquatic environment.

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HÄGER & WERKEN GmbH & Co. KG